



VISTA | APEX

Best-Etch™



REF 502115

## Best-Etch™

37% Phosphoric Acid Etching Gel

### Instructions for Use

1. Prepare tooth surface.
2. Isolate and dry tooth thoroughly with moisture/oil-free air.
3. Apply Best-Etch™ Gel to the prepared tooth. Etch enamel for 15 seconds, etch dentin for 10 seconds. Never leave etch on tooth more than 60 seconds. Lightly agitate the etchant during its application to enhance its action.
4. Rinse thoroughly with water.
5. If using a bonding adhesive, follow the manufacturers' instructions to determine if the surface of the tooth is to remain moist or be thoroughly dry.

**Storage:** Refrigeration is not required.

- Store at room temperature (22 °C/72 °F).

### Warning:

- Always replace cap after use.
- Avoid contact with soft tissue, skin and eyes.
- Best-Etch™ contains phosphoric acid, which is harmful. In case of contact with eyes, seek immediate medical attention.



May be corrosive to metals. Causes severe skin burns and eye damage. Wash thoroughly after handling. Wear protective gloves/clothing/eye protection/face protection. Absorb spillage to prevent material damage. Store in a cool, dry, well-ventilated area away from heat, direct sunlight and all sources of ignition. Store away from incompatible materials.



Keep container closed to prevent contamination. Dispose of contents and container in accordance with local and national regulations. **FIRST AID: IF ON SKIN (or hair):** Take off immediately all contaminated clothing. Rinse skin with water/shower. Wash contaminated clothing before reuse. **IF SWALLOWED:** Rinse mouth. Do NOT induce vomiting. **IF INHALED:** Remove victim to fresh air and keep at rest in a position comfortable for breathing. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician. **FIRE:** Not combustible. Use extinguishing media suitable for surrounding. Under fire conditions, toxic, corrosive fumes are emitted.



Inter-Med, Inc.  
2200 South St.  
Racine, WI 53404 USA  
Tel: +1-262-636-9755

Made in USA

Rx ONLY

502115-I-V-ENG (7)



Emergo Europe

Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
Tel: +31 (0)70 345 8570  
Fax: +31 (0)70 346 7299