

REF 317001

REF 317011

## Indications:

EDTA solution (17%); necessary for the removal of the smear layer and dentin mud.

## Instructions for use:

1. Select desired size syringe and irrigation tip.
2. Fill syringe with EDTA (10mL recommended).
3. Irrigate as desired.
4. Remove and dispose of irrigating tip from the syringe.
5. Dispose of syringe after use.

## Use and Storage:

- For dental use only.
- This medical device should not be used after the expiration date.

## Warnings:

- \*\* Not to be used as an oral rinse. \*\*
- Always replace cap after use.
  - Avoid contact with skin and eyes.
  - Keep out of the reach of children.
  - Store at room temperature.
  - Not for injection.

REF 317001

REF 317011



## WARNING

May be harmful if swallowed.  
Causes mild skin irritation.  
Causes eye irritation.

Wash hands thoroughly after handling. Wear protective gloves and eye wear. Follow all government regulations for waste disposal. Prevent release into the environment if possible. Do not flush waste into sewer or waterways.

**FIRST AID: IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do so. Continue rinsing. **IF ON SKIN:** Wash with plenty of soap and water.



## Rx ONLY



**Inter-Med, Inc.**  
2200 South St.,  
Racine, WI 53404  
Tel: +1-262-636-9755

**Made in USA**  
317011-I-V-08-ENG (2)



**Emergo Europe**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands