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Patient Instructions for Use:

1. Brush teeth before each application of Fluorescent™.
2. Remove syringe cap and fill bleaching gel into trays using 1/3 to 1/2 of syringe.
3. Place whitening gel in the face-side of each tooth space in the deepest part of the tray. Use only tear-drop size of gel.
4. Gently place tray over teeth and allow any excess gel to escape.
5. Remove any excess gel in gum area with toothbrush.
6. Rinse mouth and do not swallow.
7. Tray can be worn at any time during the day. Some sensitivity is expected. If severe sensitivity occurs, stop usage and discuss with your doctor.
8. After each application, remove tray and rinse with cool water. Use toothbrush to remove gel.
9. When not using tray, store in case provided.

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 **WARNING**
Causes serious eye damage. Avoid contact with the skin and eyes. Wash thoroughly after handling. Prevent entry into waterways, sewers, basements or confined areas. Contain and collect spillage with non-combustible absorbent material and place in container for disposal according to local/national regulations. Not considered a hazardous waste according to Federal regulations. This material could become hazardous if the discarded material comes in contact with a hazardous waste. Clean any contaminated surface thoroughly. **FIRST AID: IF IN EYES:** Rinse immediately with plenty of water, also under the eyelids, for at least 15 minutes. Keep eye wide open while rinsing. **IF ON SKIN:** Wash off immediately with soap and plenty of water. **IF INGESTED:** Do not induce vomiting. Rinse mouth, Drink plenty of water. **Specific treatment:** Immediately call a Poison Control Center or doctor/physician. **FIRE: In case of fire:** Use extinguishing measures that are appropriate to local circumstances and the surrounding environment. **CAUTION:** Use of water spray when fighting fire may be inefficient.

FLUORESCENT™

Tooth Whitening System

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Whitening Precautions:

1. Household bleach should never be used to bleach teeth.
2. If pregnant, do not use Fluorescent™.
3. Keep out of direct sunlight and heat. Do not freeze.
4. Teeth with crowns or fillings do not react to whitening gel.
5. Avoid citrus fruits and juice during the process, since it may increase sensitivity.
6. Contact your dentist for any questions regarding usage of Fluorescent™.
7. Avoid eating, drinking or smoking while wearing custom trays.
8. Coffee, tobacco, and other products can re-stain your teeth.
Ask your doctor about touching up teeth with Fluorescent™ Refill Pack.
9. Do not swallow whitening gel.
10. Tooth area along the gum line may take longer to whiten, as this area is naturally darker.
11. If gum or tooth sensitivity occurs, contact your dentist for instructions.
12. Continue regular check-ups and cleanings after whitening process.
13. A purple color may appear in the tray after bleaching teeth.
This is due to silver fillings and has no effect on the bleaching process.
14. Dispose remainder of whitening gel once the process is complete.
15. Keep Fluorescent™ out of reach of children.

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RxOnly — For dental use only.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

VISTA DENTAL PRODUCTS

2200 South St., Racine, WI 53404

Toll Free 1-877-418-4782 1-262-636-9755

Appel gratuit +1-877-418-4782 +1-262-636-9755

www.vista-dental.com Made in USA Fabriqu  aux Etats-Unis

504300P-I-V-71-ENGFRA (4)