

## NiTi SuperFlex™ Tips Patented

- Instructions for use:**
- 1) Rotate needle tip hub firmly on prefilled luer lock syringe
  - 2) Gently depress plunger to deliver solution
  - 3) Remove tip from syringe

### **To clean tip: Do not allow tips to exceed 125°C.**

1. Soak in a mild, pH-neutral detergent until cleaning can be performed.
2. Rinse tips under a hard stream of water to remove debris on the outside surface of the tip.
3. Flush tip with a syringe filled with distilled water or alcohol to clear the tip.
4. Clean tip in an ultrasonic unit according to unit manufacturer's instructions using a low-foaming pH-neutral detergent. Use distilled water if possible. Do not allow tips to contact any other metals. Thoroughly rinse tips according to instructions supplied by detergent manufacturer.
5. Wrap tips for sterilization with wraps washed in pH-neutral detergents or disinfectants. Do not use commercial wraps with sterilization indicators that leave stains. Do not mix tips with other metals during autoclaving, or allow them to directly touch metal trays. Steam autoclave at 121°C/250°F, 15-20 psig for 30 minutes.



**Inter-Med, Inc.**

2200 South St  
Racine, WI 53404 USA  
Tel: +1-262-636-9755

405107-1-02-ENG (2)

*Made in China*